



MAXIMUM pH PLUS D.S.

ALKALINE HOSPITAL INSTRUMENT CLEANER & PRE-SOAK CONCENTRATE FOR USE IN WASHER DISINFECTORS, AND FLEXIBLE ENDOSCOPE PROCESSING



PRODUCT PROPERTIES

- Alkaline Detergent • Excellent compatibility with soft metals
- Concentrated allowing additional savings on usage, storage & transport

MATERIAL COMPATIBILITY

Maximum pH Plus D.S. Detergent is designed for the manual and automated cleaning or re-usable medical instruments and devices

- Stainless Steel • Aluminium • Anodised Aluminium • Chromium Plated Material • Synthetic Materials
- Rubber • Latex • Glass • Ceramics • Copper • Brass • Metal Alloys

When used correctly in the correct dilutions no material damage is to be expected

AREAS OF APPLICATION

- For use in washer disinfectors • For use in ultrasonic baths
- For use with flexible endoscope processing

COMPOSITION

Contains amongst other ingredients; - Disodium Metasilicate Cas No 6834-92-0, pH 12-14 Typical.

DIRECTIONS FOR USE

When changing containers, wear rubber gloves and goggles and plastic apron. Place feed tube into bottom of container and check **AUTOMATIC FEED EQUIPMENT** is switched on.

Dilution for Use: Inject at 2mls per litre of wash water. **Temperature Range:** Between 45-60 Deg C.

