







Hi, my name is Saffi, I'm the **SafMed** Product Specialist

My role is to assist anyone using **SafMed** Surgical products to identify & understand the value they bring into the Operating Theatre.

## Symbols used in this document

	<p><b>Instructions for Use</b></p> <p>Always consult IFU's (equipment manuals or package inserts) for detailed manufacturer's instructions for use</p>
	<p><b>Quality Assurance</b></p> <p>Anywhere a standard is referred to or the product can form part of your quality process</p>
	<p><b>Did you know?</b></p> <p>Useful information</p>
	<p><b>Take Note!</b></p> <p>Take special note of interactions</p>

## Himax



The **Himax** is equipped with a powerful motorized back/leg section that provides easy patient

### Why do I need this table?

- The Himax surgical table is a mobile electro-hydraulically operated surgical table that is designed to support all general surgical procedures including cardiac and vascular, endoscopic, gynaecology, urology, nephrectomy, neurology, ophthalmologic, and orthopaedics

### Features of the table

- This table features a powered lateral tilt, Trendelenburg/reverse Trendelenburg, leg section, backrest, and adjustable height function
- This table is designed to safely function with a 270kg patient

### How do I use the Himax?

- During patient positioning, moving the different sections of the table-top and using extensions and accessories can cause pinch points to appear which can injure the patient and surgical staff. Care should be taken when using the table
- **Lock the table** to the floor **before positioning the patient** and starting the surgical procedure
- When moving the table with a patient on it, it is strongly advised to use side barriers to hold the patient in place
- Always plug the table in at the end of the day
- Ensure that the remote control is also properly connected to charge its battery
- Never leave anything on the foot piece/base or next to the column of the table
- Ensure proper cleaning between cases and at the end of the day
- Remember to clean the accessories before putting it on a trolley or into storage



**Do not use chlorine-based** products or other products containing chlorine derivatives; these can cause excessive **corrosion**

- Ensure the theatre tables brakes are on when transferring a patient
- Service your theatre table at the recommended intervals to ensure continued functioning



**Always refer to the instructions for use (manual)**