



Your solutions partner for **30 YEARS**

**BROWNE
FORMALDEHYDE
CONTROL SPOTS**

Instructions for use



Product description

The Browne Formaldehyde self-adhesive indicator spots are Type 1 indicators designed to be placed on the outside of every pack being sterilized in a low temperature formaldehyde (LTSF) sterilizer. Conforms with ISO 11140-1.

Pack size

Description	Size
Browne Formaldehyde Control Spots	250/Per Roll

Indications for use

The Browne Formaldehyde indicator spots change color after processing allowing differentiation between processed and unprocessed loads.

Safe Storage

The indicator and packaging contain no added heavy metals, no known carcinogens, and no added rubber latex. The indicator is considered safe when used under normal conditions. The indicators are designed to be used in low temperature steam / formaldehyde (LTSF) sterilizer.

When used as intended, the indicator does not release any substances known to be toxic in sufficient quantities to cause a health hazard, during or after the sterilization process for which it is designated, in accordance with ISO 11140-1:2005, 5.9

Recommended Storage

Cool, dark, dry conditions before and after use: 0 - 30 C 30 – 60 % RH

Disposal

Treat unwanted indicators and packaging in the same way as normal paper waste

Shelf Life

18 months from date of manufacture

Direction for use

Step 1:

The self adhesive spots should be placed on the outside of every pack being sterilized. The spots change color after passing through the process allowing differentiation between processed and unprocessed loads.

Step 2:

The spots change color after passing through the process allowing differentiation between processed and unprocessed loads.