



Your solutions partner for **30 YEARS**

**STERIS PRE-KLENZ™
INSTRUMENT TRANSPORT
GEL**

Instructions for use

Product description

STERIS PRE-KLENZ™ Instrument Transport Gel is formulated in a gel/surfactant combination to initiate the cleaning process of surgical instrumentation at the point of use and to maintain moisture during transportation.

PLEASE NOTE: Always refer to the MIFU of a surgical instrument for compatibility with cleaners & detergents.

Container Size

Description	Size
PRE-KLENZ™	650ml

FEATURES

- * Safe to spray in the operating room
- * Decreased potential for cross-contamination in the environment
- * Biodegradable surfactants (meets EU648 guidelines)
- * Decreases spills and reduces the amount of soiled instruments handled

BENEFITS

- * Can reduce cumbersome manual, pre-cleaning steps
- * No significant weight added unlike traditional pre-soak trays
- * Broad material compatibility
- * Will not stain instruments or equipment
- * Gel formulation allows you to see sharps, unlike foam

Indications for use

PRE-KLENZ™ surgical instrument transport gel provides point of use cleaning of surgical instruments. The neutral pH, clear gel is applied on surgical instruments to keep instruments moist and initiate pre-cleaning. Spray PRE-KLENZ evenly over the surgical tray of instruments to ensure soils are evenly covered. There is no need to rinse the product prior to automated processing.

Precautions

- | | |
|-------------------------------|--|
| Precautions for safe handling | : Read label before use. Avoid contact with skin, eyes and clothing. Avoid breathing mist or vapor. Keep container closed when not in use. Work in a well-ventilated area. Wash hands and other exposed areas with mild soap and water before eating, drinking or smoking and when leaving work. |
| Hygiene measures | : Wash hands thoroughly after handling. Take care for general good hygiene and housekeeping. Do not eat, drink or smoke when using this product. Wash contaminated clothing before reuse. |

Storage

- | | |
|--------------------|---|
| Technical measures | : A washing facility/water for eye and skin cleaning purposes should be present. Provide local exhaust or general room ventilation. |
| Storage conditions | : Keep out of reach of children. Keep only in the original container in a cool, well ventilated place. Store at room temperature. Avoid freezing . Keep container closed when not in use. |

Direction for use



Step 1

Place soiled instruments in tray.
Ensure that all instruments are open.



Step 2

Turn nozzle to “Spray” position.



Step 3

Apply 5-10 sprays per tray.



Step 4

Instruments are now ready
for transport.