

# Restore Your Investment

## Prolystica Restore™ Descaler and Neutralizing Detergent

You invest in stainless steel instruments to take care of your surgical patients. And you invest in quality automated washing systems to take care of those instruments. Wouldn't it be nice to take care of the instruments and the washers with a single product?

Prolystica Restore Descaler and Neutralizing Detergent is a multi-purpose, all-in-one liquid detergent that protects both the instruments and the washers – helping you protect your investment – and the environment.

Before

After



STERIS Prolystica Restore™  
Descaler & Neutralizing Detergent

1C45-AW 4 Liters (1 Gal. 7.3 fl. oz.)

Prolystica Restore Descaler & Neutralizing Detergent is an all-in-one, phosphate-free, non-flaming, acidic liquid detergent for use on both automated and manual cleaning applications.

#### MULTI-USE PRODUCT BENEFITS

Benefits	Features
De-rusting / Passivating	Enhances corrosion resistance, while restoring and maintaining the passive layer integrity of stainless steel surfaces
Descaling / Descaling	Removes hard water scale deposits and rust
Neutralizing	Effectively neutralizes high alkaline detergents
Biodegradable ingredients	Environmentally friendly

#### DIRECTIONS

Automated washers – For descaling, use 1 to 2 fluid ounces per gallon of water (3 – 16 mL per Liter of water).

Pre-soak – May be used at concentrations up to 30% v/v for heavily stained stainless steel instruments.





Specially formulated  
to be friendlier to  
the environment.

## More Applications. More Benefits. *Fewer Products.*

### Protect instruments

- Restores and maintains the passive layer integrity of stainless steel surfaces
- Removes hard water scale deposits
- Removes rust (derouges) and stains

### Protect washers

- Removes hard water scale deposits (descals washer chamber)
- Saves time (virtually eliminates down time to scrub washer chamber)
- Neutralizes high alkaline detergent residue

### Protect the environment

- Environmentally friendly acids:  
**phosphate-free**
- Biodegradable ingredients
- Recyclable packaging
- Low foaming characteristics means less rinse water

### Protect your bottom line

- Decreases instrument refurbishment/ replacement costs
- Decreases washer repair and maintenance costs
- Reduces inventory of chemistries (no need to purchase separate neutralizer and descaler)

#### Ordering Information

4x4 L	IC45AW
2x5 L	IC45T4PE*
10 L	IC45T6WR

\*International ordering only



Prolystica... more from less

### Mixed Sources

Product group from well-managed forests, controlled sources and recycled wood or fiber



Printed with Soy based inks and water based coatings on FSC/Green Seal-certified paper.

Document # M3478EN.2010-12, Rev. A  
Printed 12/2010, 2500

©2010 STERIS Corporation.  
All rights reserved. Printed in USA.

 **STERIS**

STERIS Corporation  
5960 Heisley Road  
Mentor, OH 44060-1834 · USA  
440-354-2600 · 800-548-4873  
www.steris.com